


ModiVisor™

The Modifiable Mitigation Face Shield
DESIGN REGISTRATION FILED

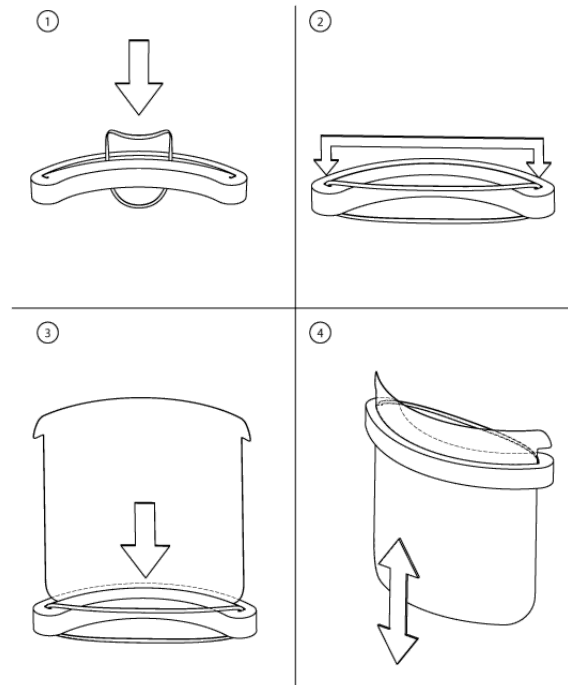

CLEANING INSTRUCTIONS:

The **ModiVisor™** is designed to easily come apart with no hidden surfaces so that the user can clean each individual component with anti-bacterial soap and water. You can fully submerge each component in a sterilization bath as the sports headpiece foam is non-porous, the elastic is non-latex / fast drying and the clear front face shield can be cleaned with a soft cloth to avoid scratching

WARNING:

PRODUCT IS INTENDED FOR MITIGATION ONLY NON-SURGICAL
PLEASE CONSULT YOUR SAFETY COMMITTEE PRIOR TO USING TO ENSURE THIS PRODUCT
MEETS THE REQUIREMENTS FOR YOUR PARTICULAR APPLICATION

ModiVisor.com

Manufactured By Plas-Tech Inc.

1-866-423-9502

www.plastechonline.com